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
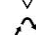
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
- Literatura**
- Weitzman, Kane and Summerbell. 1995. In P.R. Murray, E.J. Baron, M.A. Pfaller, F.C. Tenover and R.H. Tenover (ed.), Manual of Clinical Microbiology, 6th ed. American Society for Microbiology, Washington, D.C.
 - Kwon-Chung, K.J., and J.E. Bennett. 1992. Medical mycology. Lea & Febiger, Philadelphia.
 - Forbes, B.A., D.F. Sahm, and A.S. Weissfeld. 2002. Bailey & Scott's diagnostic microbiology, 11th ed. Mosby, Inc., St. Louis.
 - National Committee for Clinical Laboratory Standards. 2001. Approved Guideline M29-A2. Protection of laboratory workers from occupationally acquired infections, 2nd ed. NCCLS, Wayne, Pa.
 - Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53-80.
 - U.S. Department of Health and Human Services. 1999. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 4th ed. U.S. Government Printing Office, Washington, D.C.
 - Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021-0045.
 - Ajello, L., L.K. Georg, W. Kaplan, and L. Kaufman. 1963. CDC laboratory manual for medical mycology. PHS Publication No. 994, U.S. Government Printing Office, Washington, D.C.
 - Larone. 1995. Medically important fungi: a guide to identification, 3rd ed. American Society for Microbiology, Washington, D.C.

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