Antibiotický diagnostický disk / Antibiotic diagnostics disc / Antibiotikus diagnosztikai korong

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SUMMARY AND EXPLANATION

Diagnostic disc BAC 0,04 with content of 0,04 IU of bacitracin are designed for preliminary identification of beta-haemolytic streptococcus of group A on base of sensitivity to low level of bacitracin.

PRINCIPLE

Sensitive beta-haemolytic streptococcus of group A create defined inhibition zone around disc. Other beta-haemolytic streptococci are more resistant to these lower concentrations.

DIAGNOSTIC KIT CONTAINS

- tube with diagnostic discs - min. 500 pc (Ref. 6022-500),
min. 50 pc (Ref. 6022-50)

WARNINGS AND SECURITY PRECAUTIONS

- **For in vitro diagnostics use.**
- **For professional use only**
- Follow the instructions exactly!
- All samples and inoculated products have to be considered as potentially infectious and obvious safety valid regulations must be respected while handling.
- Do not use product out of expiration.
- Before use, check if the packaging is intact. Do not use damaged kit.
- Leftovers of expired or devaluated product liquidate according to regulations of your country. Packaging materials are recyclable.

STORAGE

Keep antibiotic discs in dark at temperatures between +2 and +8 °C

REQUIRED MATERIALS

- Petri dish with blood agar
- tampons, loops, burner, tubes and other basic equipment of microbiological laboratory

RECOMMENDED PROCEDURE

Spread thick tested strain on Petri dish, eventually spread it with tampon soaked in suspension of tested strain in saline. Place sterile disc BAC 0,04 on inoculated Petri dish, press it gently, so that whole surface of disc is in contact with medium. Incubate for 18-24 h in CO₂ atmosphere at temperature 37°C.

EVALUATION AND INTERPRETATION

Inhibition zone ≥ 10 mm - beta-haemolytic streptococcus of gr. A (*Streptococcus pyogenes*)

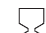
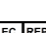
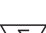
Inhibition zone < 10 mm - beta-haemolytic streptococcus of other serological group

Functionality of diagnostic discs can be verified by these control strains:

Bacterial strain	CCM / ATCC	Inhibition zone
<i>Streptococcus pyogenes</i>	4425 / -	≥ 10 mm
<i>Streptococcus sp.</i>	4685 / -	< 10 mm

CCM: Czech Collection of Microorganisms, Masaryk University Brno, Kamenice 5, 625 00 Brno, Czech Republic, tel. +420549491430, e-mail: cm@sci.muni.cz

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